

Altiven®

Graduated Compression Hosiery

Feeling better, looking good

MADE TO MEASURE

72 HOUR DESPATCH*



Features

- ▶ Manufactured to European Standards – Class 1, 2 & 3
- ▶ 72 hour despatch*
- ▶ Low compression available for night use
- ▶ Two colour options – Black & Soft Beige
- ▶ Stockings and tights available with closed or open toe
- ▶ Arm sleeves available with elbow pouch
- ▶ Latex Free
- ▶ Made in the UK



Indications

CLASS 1:
18-21mmHg Early or mild oedema where the size, shape and condition have stabilised.


CLASS 2:
23-32mmHg Mild to severe oedema where there may be some shape distortion. Long-term maintenance therapy.

CLASS 3:
36-46mmHg Severe oedema and reduction therapy.

Contra-indications

- ▶ Peripheral vascular disease
- ▶ Significant arterial disease
- ▶ Congestive cardiac failure or severe heart disease
- ▶ Diabetes (unless under medical supervision)
- ▶ Severe infection of the veins
- ▶ ABPI below 0.8 or above 1.2 (unless following a specific referral and under the supervision of a healthcare professional)

How to prescribe Altiven Made to Measure

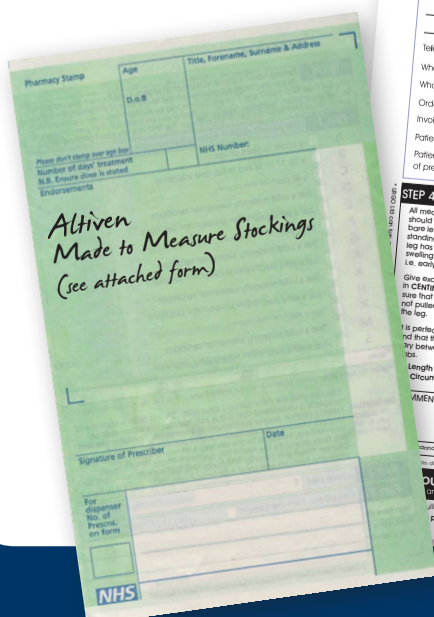

European Standard AVAILABLE ON EUROPEAN TAHOFF PFI 10

ORDER FORM
Circular Knit
Made to Measure Hosiery

ALL APPROPRIATE FIELDS MUST BE COMPLETED OTHERWISE YOUR ORDER CAN NOT BE PROCESSED

STEP 1 PHARMACY TO COMPLETE ONLY

Order Date: _____
Name of Pharmacy / Hospital: _____
Address: _____
Post Code: _____
Telephone Number: _____
Wholesaler Name: _____
Wholesaler Account No: _____
Order No: _____
Invoice to: _____
Patient's Name: _____
Patient's Reference Number (if known in the case of previous orders): _____

STEP 2 Please complete ALL sections and tick the appropriate box:

Section 1 Compression:
Low compression (14-18 mmHg)
Class 1 (18-21 mmHg)
Class 2 (23-32 mmHg)
Class 3 (36-46 mmHg)

Section 2 Style:
Below knee Thigh length
Knee to floor Full length
Foot Full length

Section 3 Insole: Full length Black
Section 4 Foot: Closed toe Open toe

STEP 3 Write quantity in SINGLES for each leg.
Must be completed with numbers or zero in each box:
Left Leg Right Leg

STEP 4

All measurements should be taken on the same leg with the patient standing and one leg has been raised and resting in a 90° minimum (i.e. scaly morning).
Give waist measurements in CENTIMETRES making sure that the tape is not pulled tight around the leg.
If perfectly normal to not take the measurements for between left and right leg.


Measurements:
Circumference: _____
Length: _____

	Thigh		Below Knee	
	Left	Right	Left	Right
CT (waist) tights only				
CH (hips)				
CG (right)				
CG (left)				
CD (below knee)				
CC (calf)				
CB (ankle)				
CA (open toe)				
LA (closed toe)				

Order to URGO Limited on 01509 501 721
Loughborough Road, Shepshed, Loughborough LE12 9JG
T: 01509 501720 F: 01509 501721

Measuring Guide

Stockings & Tights

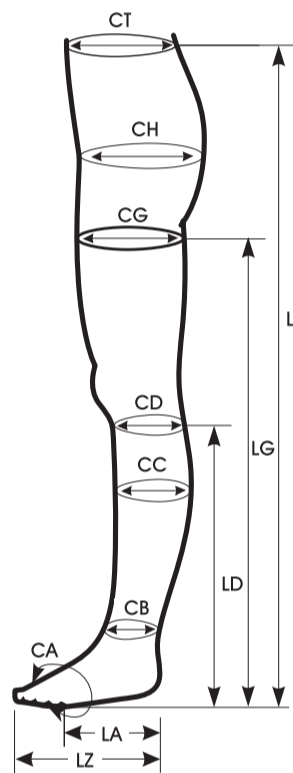


Circumference

	Tights	Thigh Length	Below Knee
Waist (CT)	✓		
Hips (CH)	✓		
Thigh (CG)	✓	✓	
Below knee (CD)	✓	✓	✓
Calf (CC)	✓	✓	✓
Ankle (CB)	✓	✓	✓
Foot (CA)	✓	✓	✓

Length

	Tights	Thigh Length	Below Knee
Waist to Floor (LT)	✓		
Thigh Length (LG)	✓	✓	
Knee to Floor (LD)	✓	✓	✓
Foot (LA open toe, LZ closed toe)	✓	✓	✓

Arm Sleeves

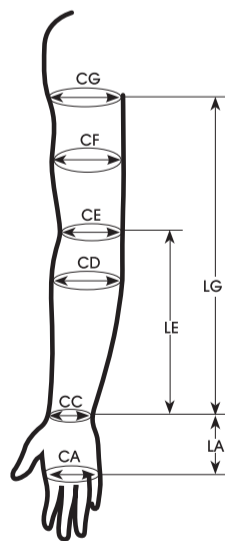

Circumference

	Axilla Wrist	Axilla Mitten
Axilla (CG)	✓	✓
Bicep (CF)	✓	✓
Elbow (CE)	✓	✓
Forearm (CD)	✓	✓
Wrist (CC)	✓	✓
Palm (CA)		✓

Length

	Axilla Wrist	Axilla Mitten
Axilla-Wrist (LG)	✓	✓
Elbow-Wrist (LE)	✓	✓
Wrist-Base of Index Finger (LA)		✓

*Altimed Ltd will despatch orders within 72 hours of receipt subject to forces beyond its control

Altiven is a registered trademark of Altimed Ltd.

Lycra is a registered trademark of Invista.

Altimed, an Urgo Medical Company, Sullington Road, Shepshed, Loughborough LE12 9JG.
T: 01509 501720 F: 01509 501721

AltiMed
An Urgo Medical Company