



British Standard
Compression Hosiery

Available
on
FP10/GP10

Altiform Made to Measure compression stockings are individually made for each patient and each leg. They fit well, are comfortable and encourage a real sense of ownership. This can improve concordance and enhance patient quality of life.


'I love my Altiform Made to Measure stockings because I know they are made especially for me!'



Benefits of Altiform Made to Measure

- Made especially for each patient to provide a superior fit
- Unique patient identification number sewn into the sole for easy re-ordering
- Soft, silky material prevents clothing from sticking
- Range of styles and colours for patient choice
- Delivered quickly with a 72 hour despatch* promise
- Latex Free

Available Styles

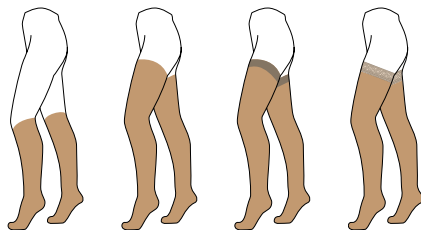

Below Knee Thigh Length Hold ups Plain Hold ups Lace

Available in Black and Soft Beige with open or closed toe.

Altiform is manufactured
in the UK

How to prescribe Altiform Made to Measure

Made to Measure order forms are available from www.altimed.co.uk or by contacting Customer Services on **01509 501720**

Altiform Made to Measure stockings (as per attached form)

Altiform MADE TO MEASURE

British Standard AVAILABLE ON DRUG TARIFF FP10/GP10

It is essential that all sections are completed fully, otherwise your order cannot be processed

PATIENT/CLINICIAN TO COMPLETE

Order Date: _____ Date of Prescription: _____ Prescription Class: _____ Requiring Group: _____
 Name and Address of Pharmacy/Hospital: _____ Name of Altiform Order Group/Pharmacy Class: _____
 Patient Name: _____ Patient No: _____
 Telephone No: _____ Patient's Previous Reference No. (if known): _____

Measurements completed by: _____
 Name: _____ Position: _____
 Date: _____

Style: Below Knee Thigh Length Hold ups Plain Hold ups Lace

Compression: Class 1 (18-21mmHg) Class 2 (23-30mmHg) Class 3 (30-40mmHg) Class 4 (40-50mmHg)

Quantity (lengths): _____
 Feet: None Both None None None

High Lengths: 15cm 20cm 25cm 30cm 35cm 40cm 45cm 50cm

Other Size: 15cm 20cm 25cm 30cm 35cm 40cm 45cm 50cm

Measurements: Ankle Calf Thigh Leg Foot Arch Heel Ball Instep Toe Arch Heel Ball Instep Toe

Comments: _____

Altiform MADE TO MEASURE

For further information visit www.altimed.co.uk or email enquiries@altimed.co.uk
 For further information call 01509 501720
 © Altiform 2010

*Subject to forces beyond our control



British Standard Compression Hosiery



Altiform British Standard compression hosiery is also available in stock sizes.

Measuring Guide


	A: Mid-thigh	B: Calf	C: Ankle	D: Foot
Small	41 – 54	32 – 38	19 – 25.5	20 – 26
Medium	44 – 58	34 – 40.5	21 – 27.5	22 – 28
Large	47 – 61	36 – 43.5	23 – 29.5	24 – 31
XLarge	50 – 64	38 – 47	25 – 32	26 – 33

Measurements in cms

Available Styles

	BELOW KNEE		THIGH	
	Open Toe	Closed Toe	Open Toe	Closed Toe
Class I	●	● ●	●	● ●
Class II	●	● ●	●	● ●
Class III	●	●	●	●

Soft Beige ● Black ●

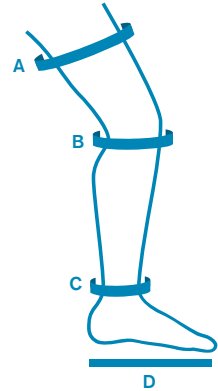

Indications

The Altiform British Standard Compression Hosiery Range is available in three classes of graduated compression.

Class I (14 – 17mmHg compression at the ankle)

Superficial or early varices, varicosis during pregnancy, swollen or aching legs and ankles. Suitable for long distance travel.

Class II (18 – 24mmHg compression at the ankle)

Varices of medium severity, venous leg ulcer treatment and prevention of recurrence, mild oedema, varicosis during pregnancy. Suitable for long distance travel.

Class III (25 – 35mmHg compression at the ankle)

Gross varices, post thrombotic venous insufficiency, gross oedema, venous leg ulcer treatment and prevention of recurrence.

Contraindications

- Peripheral Vascular Disease (PVD), confirmed with an ABPI (Doppler) reading of less than 0.8
- Acute or unstable Heart Failure
- Ischaemic Heart Disease (IHD), also called Coronary Artery Disease
- Untreated Cellulitis or Phlebitis of the Lower Limb

Proceed with caution in those patients who have:

- Diabetes
- Rheumatoid Arthritis

These patients may require referral to a specialist for advice and additional assessment. Close supervision may be necessary in line with local policies/pathways.

For more information on the Altiform British Standard Hosiery Range, please call **01509 501720** or visit **www.altimed.co.uk**



An Urgo Medical Company

Altiform is a registered trademark of Altimed Limited

Altimed, an Urgo Medical Company – Sullington Road Shepshed Loughborough LE12 9JG

01509 501 720

01509 501 721

enquiries@altimed.co.uk